


dermalogic

**LAZIO**

**SPA TREATMENT TABLE**

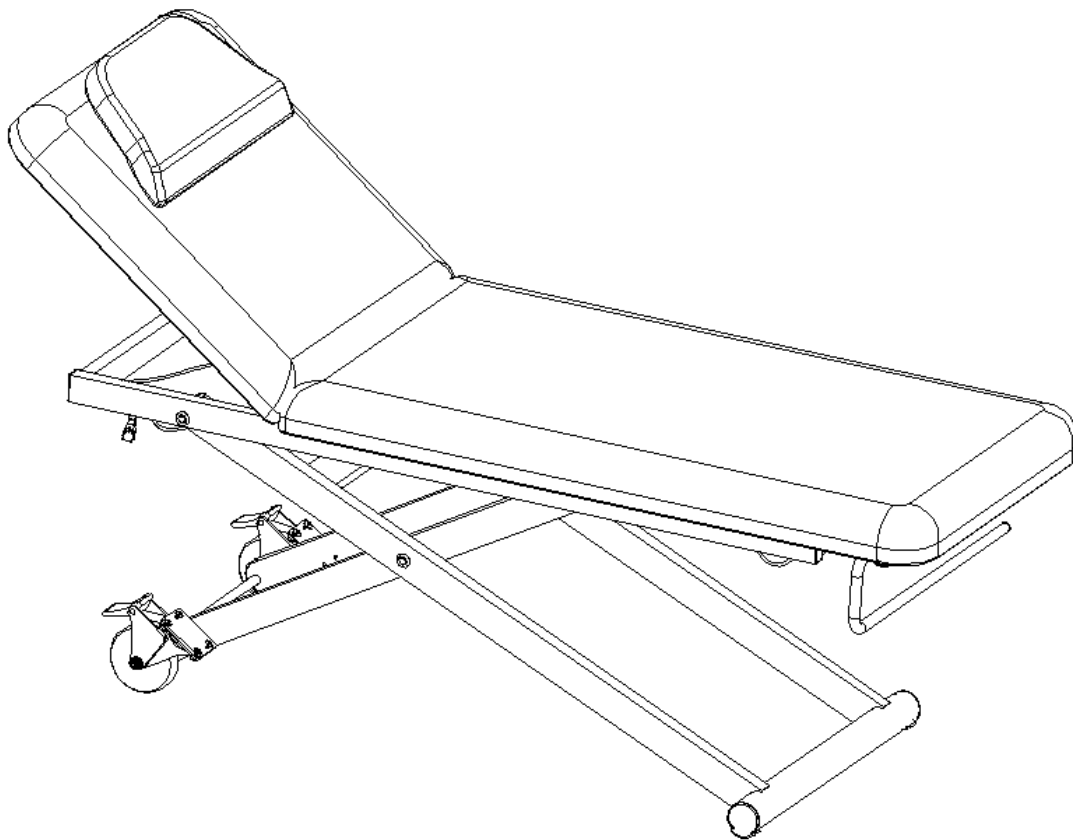

**User Manual**

HZH-FCCHR-3329E

# INTRODUCTION

Thank you for choosing Dermalogic LAZIO Spa Treatment Table. Dermalogic is an AYC's registered trademark for elegant and well-designed spa salon furnishings and appliances. We strive to bring quality, dependability and affordability when properly used.

PLEASE TAKE TIME TO READ THIS MANUAL THOROUGHLY BEFORE USE. If you have any questions or concerns with your spa treatment table, please contact our customer service and technical support at 972-926-6488 ext. 2, Monday through Friday 9AM – 5PM or email us at [service@ayc.com](mailto:service@ayc.com).

# SHIPMENT INSPECTION

Thoroughly inspect each shipment immediately upon arrival. It is very important that you notify the driver directly and note any damage or irregularities on the delivery receipt. Failure to note damage on the delivery receipt may prevent any claims you may have against the carrier.

If concealed loss or damage is found, you MUST report it to AYC within 72 hours of delivery and DO NOT DISCARD THE SHIPPING CARTON. Failure to do so constitutes acceptance of the shipment as-is and in good order and, additionally, waives any right to collect possible compensation from shipping claims.

# SPA TABLE ACCESSORIES

- Power Cord x 1
- Hand Controller x 1
- Removable Headrest x 1
- Paper Holder x 1
- Pin x 2
- B Clips x2



NOTE: Please check all the accessories that are contained after unpacking

# IMPORTANT INSTRUCTIONS

Please read the following instructions carefully before connecting, operating, or adjusting this electric spa treatment table. Retain this manual for future reference.

- **Safety First:**


This electric spa treatment table is equipped with a three-wire power cable and must be connected to a three-hole outlet with a functioning ground electrode. Ensure proper grounding to prevent electric shock during use.

- **Avoid Water Contact:**

Do not allow any plugs, buttons, or components connected to the power cable to come into contact with water or other liquids.

- **Operation Precautions:**

When using the hand control or foot control board to adjust the bed's height, do not place your hands, feet, head, or any other body parts into areas marked ①, ②, ③, and ④ in the figure below.

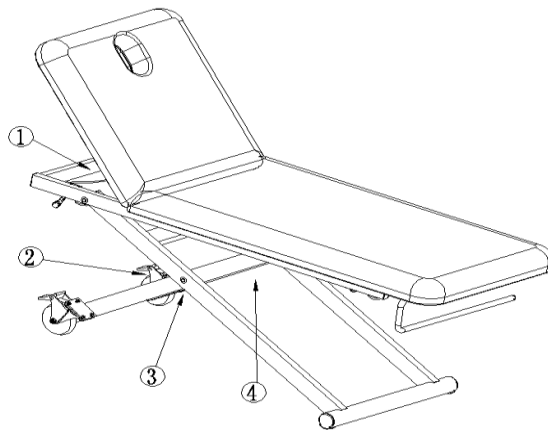

- **Damaged Components:**

Do not use the bed if any plugs, buttons, or components connected to the power cable are damaged.

- **Motor Usage Guidelines:**




The motor should operate at a frequency of **6 minutes per hour**. Do not exceed this usage limit.

- **Professional Maintenance Required:**


Maintenance or repairs must be performed only by officially trained personnel.



# SPECIFICATIONS

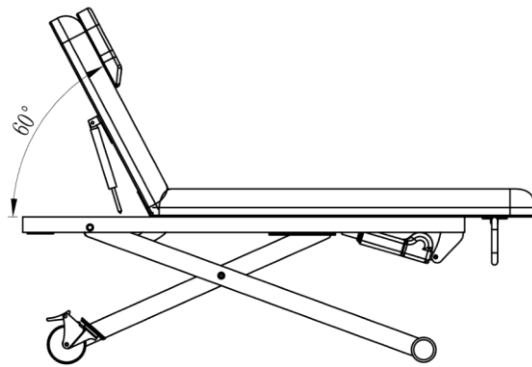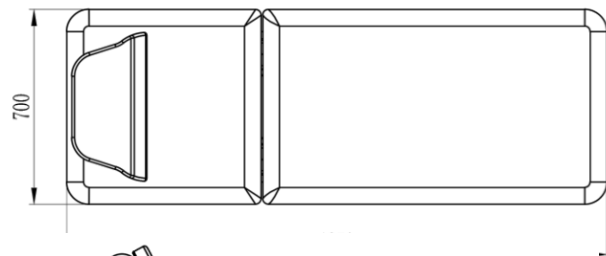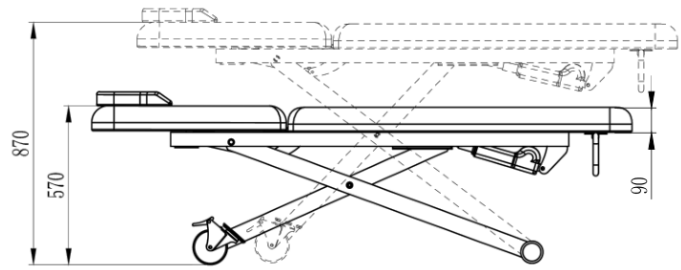- Model: HZH-FCCHR-3329E
- Dimensions: 1950×700×570mm
- Minimum Height: 570mm
- Maximum Height: 870mm
- N.W.: 50kg
- G.W.: 75kg
- Maximum loading: 200kg
- Padding: 8cm sponge
- Fabric: PU Leather
- Input voltage:  
AC100~130V/220~240V, 50/60Hz
- Output voltage: DC 24V
- Rated power: 50W

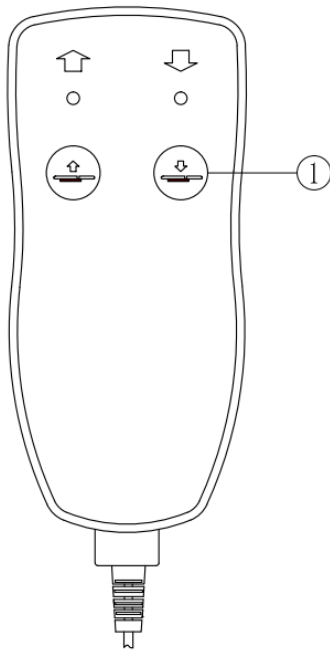


# HAND CONTROL



- ① Press  to adjust the table down  
Press  to adjust the table up


**Height Adjustment as below**

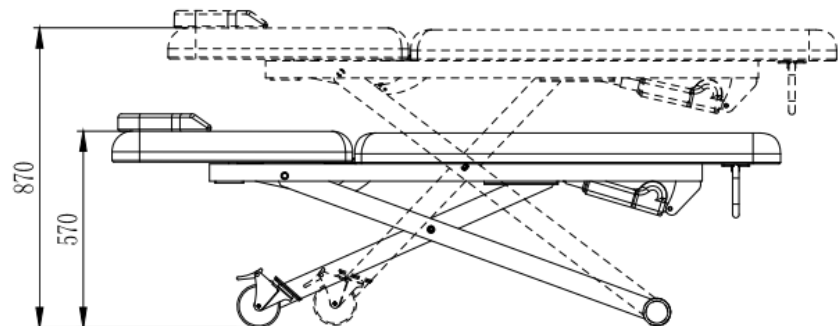

# INSTALLATION INSTRUCTIONS


## Installation of the Table Stand

1. **Align the Holes:** Position the table stand so that the mounting holes on the stand are aligned with the corresponding holes on the table frame.
2. **Insert the Pins:** Slide the pins through the aligned holes to secure the table stand in place.
3. **Attach the B Clips:** Fasten the B clips onto the pins to lock them securely.
4. **Verify Installation:** Ensure the table stand is firmly attached and does not wobble.

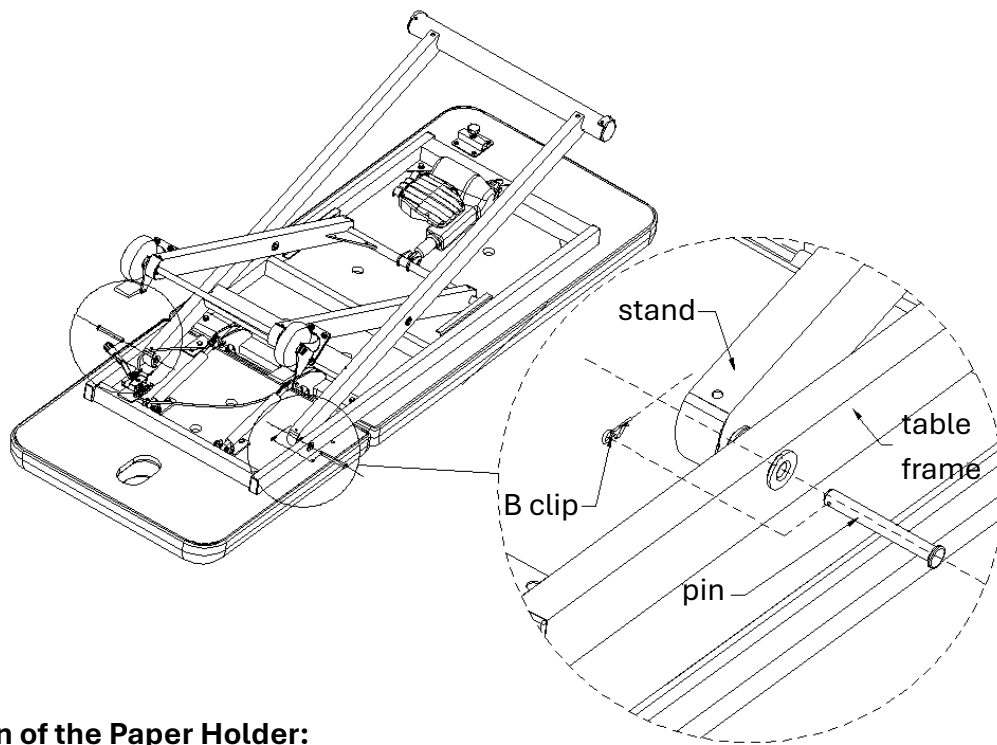

## Installation of the Paper Holder:

1. **Loosen the knob** by turning it counterclockwise.
2. **Insert the paper holder** into the designated slot or area.
3. **Tighten the knob** by turning it clockwise to secure the paper holder in place.

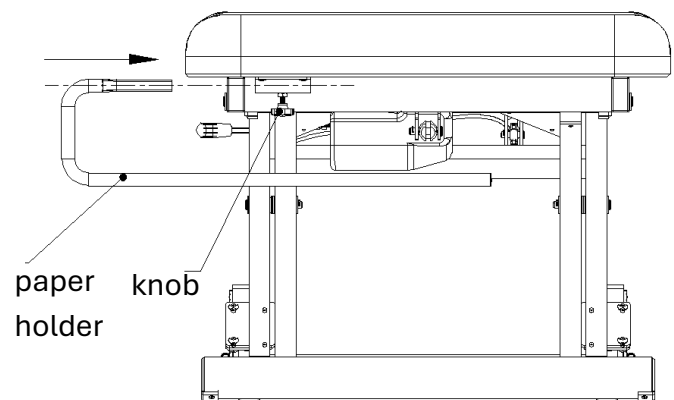


Refer to the illustration on the right for visual guidance.

# BACKREST ADJUSTMENT


## To Adjust the Backrest Up:

Pull the switch handle outward, then the backrest will rise automatically.

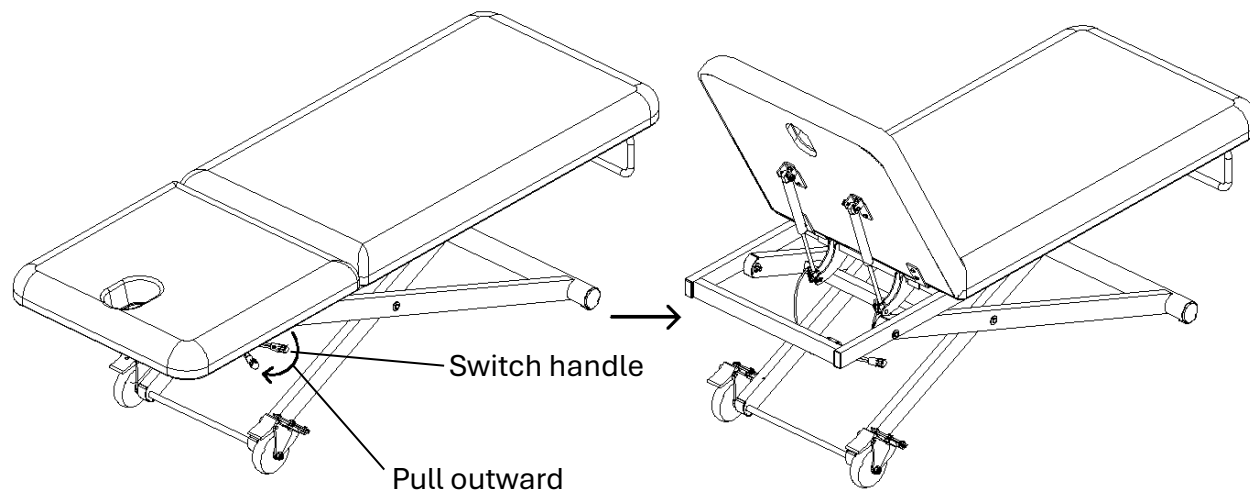


## To Adjust the Backrest Down:

Pull the switch handle outward with one hand and keep holding it. Use your other hand to press the backrest down.

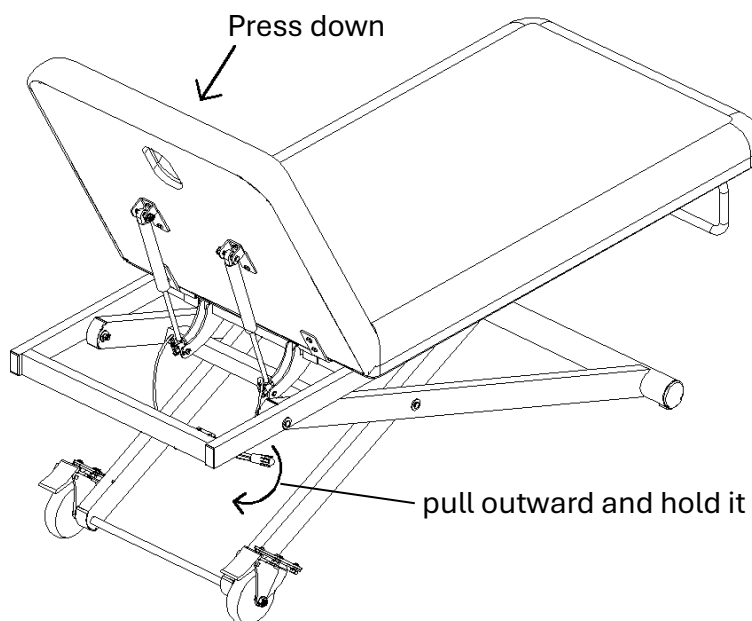

# OPERATION OF WHEELS

1. **Lift the braking system:** Ensure that the braking system on both sides of the rubber wheels is lifted or disengaged, allowing the wheels to move freely.
2. **Lift the other end of the bed:** Raise the opposite end of the bed, so the wheels are the only part in contact with the ground.
3. **Wheel movement:** Once the bed is lifted, the rubber wheels will be the only parts making contact with the floor. This allows the bed to slide more easily, conserving

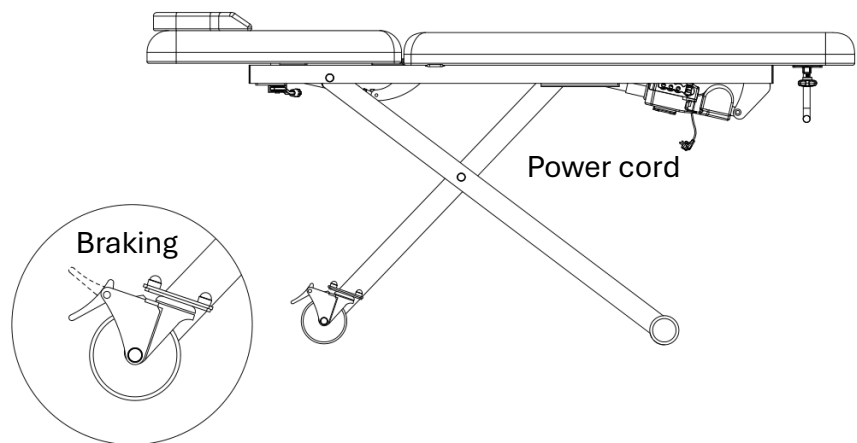

## PRECAUTION

- Maintain a distance of at least 20 cm between the control box, hand control, and high-frequency equipment to avoid interference.
- Ensure power cables and plugs are intact before use. Any damage should be addressed by professionals or service providers.
- Verify that the voltage matches the specified safety voltage. Using incorrect voltage can cause product damage.
- When in use, the product may draw a current of 10A or more. Choose a power outlet capable of handling high current and, if possible, use a dedicated outlet to prevent power failure or fire hazards.
- Avoid handling the product with wet hands to prevent electric shock.
- Do not plug in the product before completing installation.
- Position power cables to prevent tripping hazards.
- Clean the product's surface with a soft towel and diluted neutral detergent.
- Unplug the product when not in use.
- Do not disassemble the product without manufacturer authorization.

# HEALTH WARNING

Individuals with the following conditions should avoid spa treatment.

- Medical conditions, such as osteoporosis or heart-related issues.
- Pregnancy or menstruation.
- Injuries or disabilities.
- Unconsciousness or illness.
- Children under 14 years old without supervision.

**If the client experiences any pain or discomfort, discontinue the spa treatment immediately.**

# TROUBLESHOOTING

If there is an issue with the product, please follow these steps to diagnose and resolve it:

Phenomenon	Diagnosis	Operation of Elimination
<b>No response when pressing the button on the remote</b>	<ul style="list-style-type: none"><li>• The control button function has reached its maximum.</li><li>• Power plug is not connected.</li><li>• Transformer connection is loose.</li></ul>	<ul style="list-style-type: none"><li>• Try pressing a different control button.</li><li>• Disconnect and reconnect the power plug.</li><li>• Open the chassis, disconnect, and reconnect the plug.</li></ul>
<b>Noises and shaking while going up and down</b>	Screw is loose.	Tighten all screws.
<b>The light remains off even though the switch is on</b>	The fuse has burned out.	Replace the fuse.



# MAINTENANCE

- Perform daily cleaning by wiping dust with a soft, dry towel.
- Remove heavy dirt using a soft towel with diluted neutral detergent, followed by scrubbing with a dry towel.
- Avoid using benzene, diluent, gasoline, or other solvents, as they may damage the leather or corrode the product's appearance.
- Do not expose the product to strong sunlight or a moist environment, as this may cause damage.
- Use the product within the temperature range of -5°C to 50°C and humidity levels of 45% to 85%.



[www.avcllc.com](http://www.avcllc.com)

[sales@avcllc.com](mailto:sales@avcllc.com)

**972-926-6488**

**Garland, TX**